

## CLEANING GUIDE (AFTER EACH USE)



**WARNING:**  
Please refer to the IFU for ALL safety guidelines



1 Disassemble the CLT-210 Circuit



2 Rinse all parts under warm tap water



3 Prepare the ultraconcentrated dish soap solution



4 Put all parts in solution and brush each part with a soft bristled brush



5 Stir all parts to remove bubbles



6 Leave all parts in solution to soak



7 Brush each part again, with the soft bristled brush



8 Remove all parts from the solution, rinse under warm water



9 Immerse all parts in a basin of cool water



10 Remove all parts and rinse under warm water



11 Shake parts of excess water, air dry on absorbent clean surface



12 Perform a safety inspection (See Section 14.13 of the IFU for details)

## DISINFECTION GUIDE (WEEKLY)

**WARNING:**  
Please refer to the IFU for ALL safety guidelines  
Please refer to the IFU for precautions relating to the use of 3% Hydrogen Peroxide



1 Complete the CLT-210 Cleaning Guide (See Above)



2 Pour 3% hydrogen peroxide into a basin



3 Immerse all parts in the basin of hydrogen peroxide, stir to remove bubbles



4 Allow the parts to soak



5 Remove the parts from the basin, shaking them free of excess hydrogen peroxide



6 Rinse each part under warm water



7 Leave the rinsed parts to air dry on an absorbent clean surface